

# Hemodrop Blood Dispenser

A Product Line of Biomedical Polymers



**An easier, safer way to prepare differentials. Ideal usage for:**

- Hematology
- Blood bank
- Phlebotomy
- Laboratory environments



## Features and Benefits

### Save time.

Direct access to sample. No need to de-cap the blood collection tube.

### Easy to use.

The flat surface of the Hemodrop makes insertion into the tube easy and painless.

### Visibility.

The clear plastic of Hemodrop allows you to see the drop on the slide so you know when your slide is ready.

### Safe.

Hemodrop is disposable, helping to avoid any unnecessary hazards.



## The Hemodrop blood dispenser...

is designed for the safe and easy dispensing of a single drop of blood from any style of specimen tube without removing the cap. By eliminating the need to remove the cap from the specimen tube, the introduction of airborne pathogens is minimized.

### How to Use

Making differentials has never been easier. Three easy steps are all it takes for the easy dispensing of a drop of blood from a specimen tube onto a slide.

1. Insert the Hemodrop blood dispenser into the specimen tube with your thumb.
2. Turn the tube upside down and gently press down on the slide until a uniform drop of blood appears.
3. Once drop of blood appears, relax pressure and carefully lift the specimen tube from the slide.

Item #	Description	Case Pack
BMP-HEMODROP	Hemodrop blood dispenser	1,000 per case

For more information or samples, contact us:

800.253.3684 | [bmpmedical.com](http://bmpmedical.com) | [sales@bmpmedical.com](mailto:sales@bmpmedical.com)